


Sizing and fitting guide for AcuCare masks



- Use the sizing template on the packaging to determine the correct mask size.

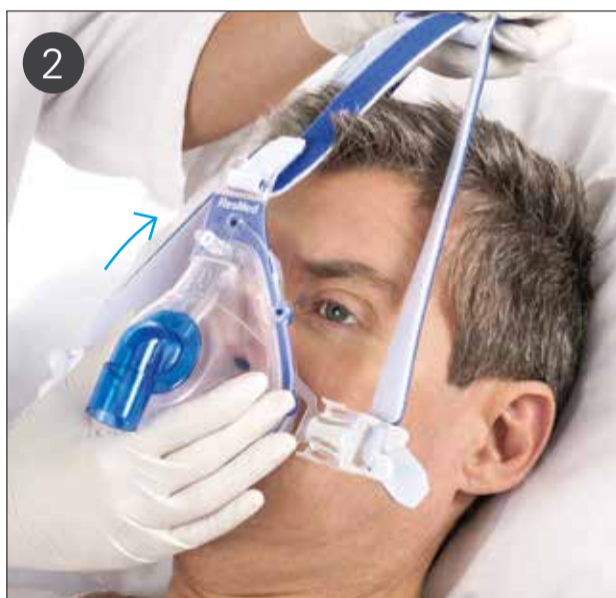

- Position the mask cushion over the nose and mouth, and pull the headgear over the head.



- While holding the mask against the face, pull each of the side straps back along the side of the face. The straps should pass below the ears.



- The side arms on the mask should bend back as the straps are tightened.



- Gently adjust the top strap.



- Start ventilation according to the ventilator system operating instructions, and connect the mask to the ventilator system.



- If air leaks around the cushion, reposition the mask on the face to achieve a better seal. Readjust the headgear straps if necessary.

Tips

Do not overtighten the straps; air fills the dual-wall cushion and provides a comfortable seal. For the best seal, the side straps should be tighter than the top strap.

Note: These are generic fitting instructions for the AcuCare mask range. Always refer to your mask user guide for details on the correct use of the mask.