


# Sizing and fitting guide for AcuCare HFNC



- Use the sizing template on the packaging to determine the correct cannula size for your patient.
- Inspect the nasal cannula before use, and replace it if any parts are damaged.



- Hold the prongs away from the nose.
- Hold the bottom strap and position it at the back of the head.

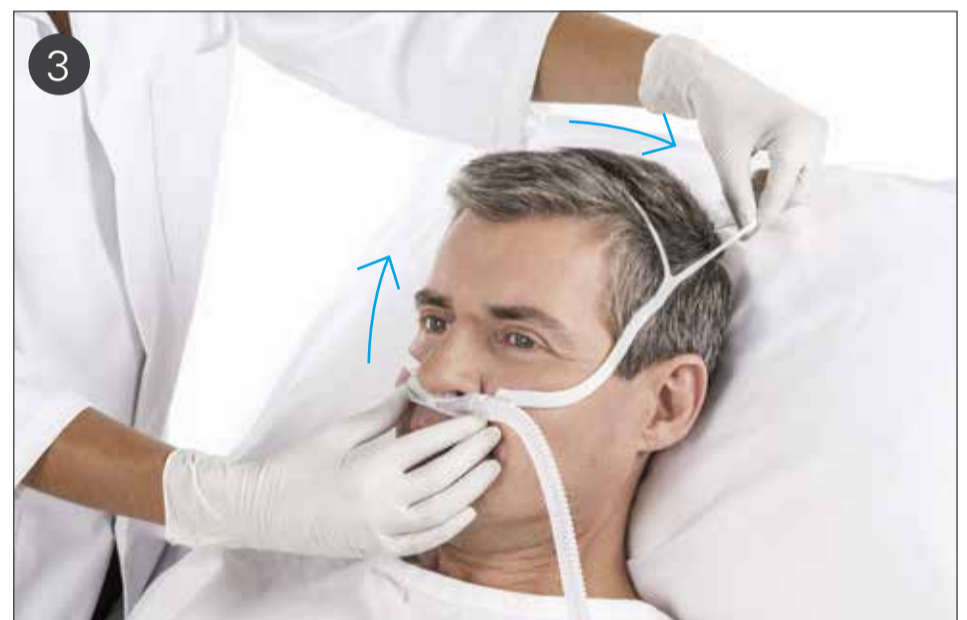

- The top strap will be positioned on the top of the head.
- Check that the prongs are in the nares, but not sealing them.
- Move the straps apart to loosen and bring them together to tighten.



- Fix the clip on the end of the short tube to your patient's garment.



- Connect the cannula to the heated humidification system tubing.

**Note:** For more details, refer to your nasal cannula user guide.